

# Identifier™

## General Screening Drugs

### TEST KITS




#### Read Before Use

As a precaution, gloves and eye protection should be worn when handling substances and using test units. Do not puncture any part of unit. Do not remove probe after breaking the ampule. In event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. Store the tubes in a dark place, not exposed to light.

#### Test Preparation

- 1** Remove the IDENTIFIER Cap from the test unit.

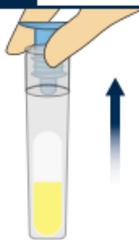


Ensure that the test tube remains upright in order to avoid the ampule falling out of the tube.

- 2** Sample the suspected substance with the IDENTIFIER Cap.



#### Test Preparation

- 3** Return the IDENTIFIER Cap to the test tube. Press firmly.

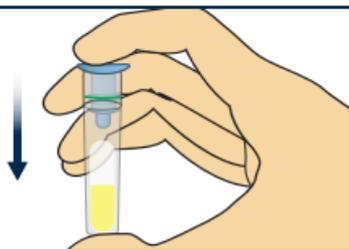


Do not continue unless Identifier Cap and black O-Ring are properly secured.

#### Identification

- 4** With thumb and forefinger squeeze the test tube until the ampule breaks.



- 5** Gently shake test up and down to ensure that the liquid touches the Cap.

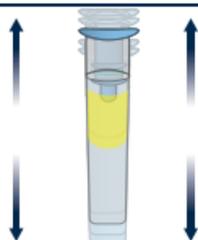

#### Results



The appearance of a purple-violet color indicates the presence of one of these suspected drugs: **Heroin, Amphetamine, Methamphetamine, Ecstasy/MDMA, Benzderone, Buphedrone, Carfentanyl, Cathinone, DMT, Fentanyl, Ketamine, Mephedrone, Methadone, MethCathinone, Methylone, Mexedrone, mCPP, MDMAI, MDPBP, MDPV, MPA, PCP, Pentedrone, Pentylone, PMA, PMMA, 2-aminoindane (2-AI), other synthetic/designer drugs.**



The appearance of a green or blue-green color indicates the presence of **Cocaine** or **Crack**.



If none of the above colors develop, none of the suspected drugs listed above are present.



#### Disposal

The test kit is safe to dispose of in the trash.

04-2021